



## Instruction for Use




Product Name:  
Nose Clip

### Product Description:

The product is made of white and blue plastic. Each nose clip features soft, grooved pads to give it a gentle, non-slip grip to ensure a proper fit, and is held comfortably in place by spring action. It is used to clamp the nostrils comfortably while making respiratory ventilation by mouth.

### Production Specification:

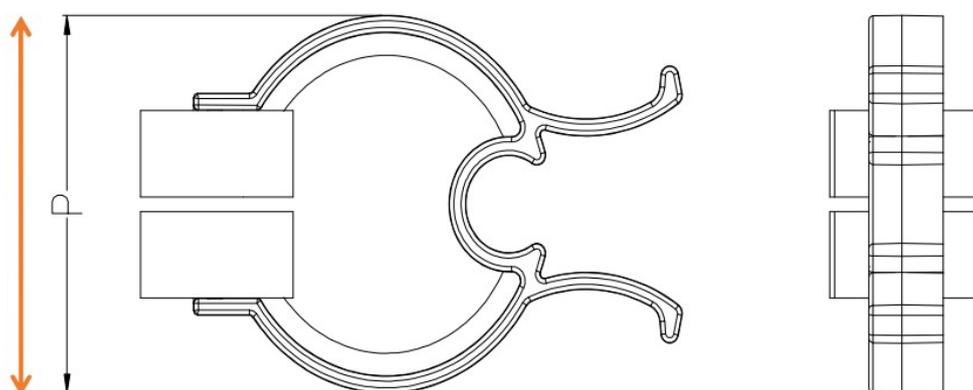

Item No.	Size	Weight
TW8501	P:40± 2mm	5.2± 0.5 g
TW8504		5.2± 0.5 g



#### Intended purpose

The product is used to block nasal ventilation while the respiratory process need.

#### Indications

The Nose Clip is intended for single patient use, which is used to block nasal ventilation while the respiratory process need.

The product has no specific indications.

#### Intended users

Qualified and trained medical staff.

#### Intended patient population

Except for users mentioned in contraindications, there is no restricted user base.

#### The combination of the device with product

The Nose Clip is not intended for combination use with other products.

#### Instruction for Use

1. Press the tail end of the nose clip with your hand to separate the nose clip cushion to the maximum space.
2. Place the separated nasal clip pads over the user's two nostrils.
3. Release the nasal clip to allow the nasal clip pad to grip the user's nostrils.

#### Cleaning instructions

Cleaning method: Soak in warm water

- a) Remove the nasal cushion of the nose clip and soak it together with the nose clip arch in warm water at 50 °C~60 °C. Ensure that the water surface covers all components.
- b) Soak for about 10 minutes and remove. Rinse with running tap water and wipe all parts around with a cotton swab.
- c) Use new medical gauze to wipe off the moisture of the components or air dry them naturally, and store them in a dry and clean place.

Do not use equipment such as hair dryers, microwaves, dryers, etc. for drying.



#### Cautions



1. Please use the nose clip under professional instructions to avoid injury caused by improper use.
2. Children must have adult supervision when using nasal clips to prevent misuse or suffocation.
3. Ensure that the size of the nose clip is suitable for your nose shape to avoid improper wearing may cause discomfort or injury.
4. Before using the nose clip, please check if it is intact and undamaged. Damaged nose clips may lose their medical effect.
5. If you feel uncomfortable wearing a nose clip, please stop using it immediately and consult the medical expert.
6. Nose clips should not be worn for a long time, and appropriate rest should be taken after use to avoid nasal compression.
7. Do not use the nose clip for unexpected purposes besides the intended use, incorrect use may cause accidents or damage.
8. This product is for repeated use by a single patient.

#### Shelf life

The shelf life is valid for three years at the room temperature.

#### Contraindication

1. Patients with nasal bleeding and soft tissue damage should use it with caution.
2. Patients undergoing nasal filling surgery should use it with caution.

#### Disclosure of residual risks

N/A

#### Storage and Transportation

The product should be stored in a cool, dry, non-corrosive gas, well ventilated, and clean environment. Do not squeeze during transportation.

Symbol



	Manufacturer
	Authorised Representative in the European community
	Importer
	Medical Devices
	Caution
	Consult instructions for use
	Do not reuse
	Non-sterile
	Catalogue number

	CE mark
	Unique Device Identification
	Batch code
	Date of manufacture, made in China
	Use by
	Keep dry
	Keep away from sunlight
	Keep upright during transport
	Single patient multiple use



## Incident Reporting

The user and/or patient shall report any serious incident that has occurred in relation to the device to Xiamen Tianzuo Medical Co., Ltd. and the competent authority of the EU Member State in which the user and/or patient is established.



Xiamen Tianzuo Medical Co., Ltd.  
Area A, 6F, No. 588-1, TongFu Road, TongAn District, Xiamen City,  
361100, Fujian Province, P. R. China



MedPath GmbH  
Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany